

Gymna ShockMaster 300

Serious about shockwave treatments



Features

1. Guided Therapy System (GTS)

Shockwave therapy is evolving constantly. Thanks to our software updates, you stay abreast of new developments:



- **Unique! patient Guided Therapy System:** the intelligent Shockmaster patient Guided Therapy System (pGTS) will automatically adapt treatment level, based on your patient's condition. Pre-programmed settings will vary after the assessment of the current pain level of the patient and the clinical investigation of the therapist.



Settings vary according to patient's condition


- **Total therapy plan:** every application is divided in different steps: pGTS, tendon treatment, trigger point treatment, muscle treatment with D-Actor, muscle treatment with V-Actor.




- Treatment parameters and applications are adapted on a regular base, according the latest scientific information.
- All information on a specific sequence is available as a pop-up message, without the need to leave this application. No time loss and all practical guidelines easy accessible.
- **Unique!** Different applicators in the same sequence, might contain different parameter settings, ensuring a safe and efficient treatment.

2 Intuitive user interface

- **Easy application search:** find your application in the library, in one of the submenus or through the body area. Submenus are based on the affected tissue.

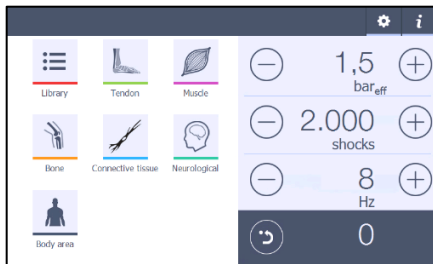

- **Unique:** marked treatment area and referred pain with trigger point

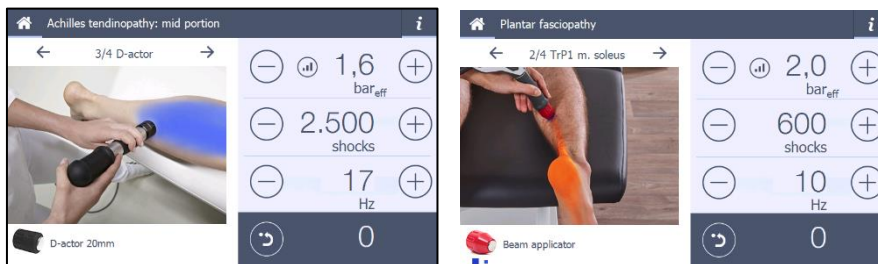

- **Safety restriction:** preventing the use of an inappropriate handpiece
- **Pop-up** after 1.25 million shocks, when revision is needed
- **Contra indications:** easy check list before each treatment

3 Handpiece short

- Ergonomic design: lightweight and well balanced
- Integrated soft grip materials
- Extended energy range: starting level at 0,3bar_{eff}
- Compatible with fascia and spine applicators
- Backwards compatibility: the short handpiece may be used on all ShockMaster devices provided that the mandatory software update is performed



4 User friendly

- Easy to transport thanks to the custom made carry bag



- Silent air compressor, **no maintenance, no emptying of the condensation bottle**

- **Unique automatic condensation**
- Compressor switches on only during treatment
- Rapid software update via USB port



5 Gymna Club

Free subscription on the Gymna club:

Extra information about shockwave therapy with the Gymna ShockMaster: literature, technical support, software updates free of charge, extra marketing material

Technical specifications

Characteristics

Dimensions (H x D x W cm)	18 x 38 x 42
Weight (kg)	11
Compressor	Integrated air compressor, silent, vibration free No maintenance with the automatic condensation
Mobile	Portable, with the custom made carrying bag
Connections	1
Touch screen	7inch Full color 180° view

User interface

Free mode	Plug and play
Applications list	41 pre-programmed settings
Total therapy plan	Every application is divided in different sequences
Body area	Choose applications via a body area
Marked treatment area	Treatment area marked on the pictures during treatment Trigger point referred pain application
Quick start	Tendon applications Muscle applications Bone applications Connective tissue applications Neurological applications
Pre-set counter	Supported via illuminated bar
Software update	Via USB port

Performance

Frequency range	0,5 - 17 Hz (in steps of 0,1 Hz)
Max frequency with V-Actor	31 Hz
Pressure range	0,3 - 4 bar _{eff} (in steps of 0,1 bar _{eff})

Handpiece and applicators

On/off function	Electronic No air cooling
Weight (kg)	0,45 kg
Ergonomics	Integrated soft rubber protection
V-Actor vibration therapy	Available
Revision kit (standard)	4 revisions Do it yourself possibility
Applicators (standard)	Beam applicator D-actor 20 mm applicator Deep impact applicator

Guided Therapy System

GTS software	Intuitive layout
GTS pre-programmed settings	Complete treatment info patient Guided Therapy System
pGTS	patient Guided Therapy System
Contraindications	Contraindications list
Safety restrictions	Preventing wrong handpiece connection
Revision message	After 1.25 million shocks

Marketing material

Waiting room Poster
Certificate
Application guide

Support

Gymna Club	Scientific literature, technical support, free software updates, marketing material, letter for the doctor
------------	---

Conformity

Classification according to	
MDD 93/42/EEC:	IIa
MDR SOR/98-282:	II
Electrical protection class	I
Applied parts type	type B
Protection against the ingress of water:	IPX1
Notified body	CE0344

Ordering information

<u>Article</u>	<u>Article number</u>
Gymna ShockMaster 300	342.300

Standard included:

Device Gymna ShockMaster 300, RSW handpiece short, beam applicator, deep impact applicator, D-actor 20mm applicator, 4 revision sets (2 projectiles and tubes inside hand piece box + 1 revision kit RSW handpiece short), 1 protective antiskid mat, 1 bottle physio care contact neutral hydrogel 500ml, 1 tissue box, 1 waiting room poster, 1 certificate and 1 applications guide

Accessories:

Gymna ShockMaster RSW handpiece short	342.716
Gymna ShockMaster body replacement set RSW handpiece short	342.749
Gymna ShockMaster revision kit RSW handpiece	309.771
Gymna ShockMaster revision kit RSW handpiece short	342.705
Gymna ShockMaster V-actor	338.239
Gymna ShockMaster fascia applicator set	342.727
Gymna ShockMaster spine applicator set	342.738
Gymna ShockMaster classic 15mm applicator	338.613
Gymna ShockMaster beam applicator	338.618
Gymna ShockMaster deep impact applicator	338.624
Gymna ShockMaster focus lens applicator	338.630
Gymna ShockMaster atlas applicator	342.760
Gymna ShockMaster acupuncture 6mm applicator	338.602
Gymna ShockMaster ceramic applicator	342.771
Gymna ShockMaster trigger 10 mm applicator	342.782
Gymna ShockMaster D-actor 20mm applicator	338.646
Gymna ShockMaster D-actor 35mm applicator	338.657
Gymna ShockMaster D-actor 20 mm "T" applicator	342.793
Gymna Physio Care Contact neutral hydrogel 500 ml	341.088
Gymna Physio Care Contact neutral hydrogel 5 L	341.099
Gymna ShockMaster protective anti skid mat	338.217
Gymna ShockMaster V-actor 25mm applicator	338.668
Gymna ShockMaster V-actor 40mm applicator	338.679
Gymna ShockMaster Poster	342.551
Gymna ShockMaster 500 trolley	338.960
Gymna ShockMaster 300 custom made carrying bag	338.316
Gymna ShockMaster application guide EN	342.673